

User Manual

MagniLink Read

LVI Low Vision International

Verkstadsgatan 5

352 46 Växjö

Phone: +46 470 727700

Fax: +46 470 727725

E-mail: info@lvi.se

Internet: www.lvi.se

LIST OF CONTENTS

1.	INTRODUCTION.....	4
1.1	System Requirements*	4
2.	INSTALLATION	5
2.1	Software download	5
2.2	Installation	6
3.	LICENSING	11
3.1	Connected to the Internet	12
3.2	Not connected to the Internet	13
4.	AUTO UPDATE	14
5.	USER INTERFACE	15
5.1	Command icons	17
6.	WORKING WITH MAGNILINK READ	18
6.1	Starting the program	18
6.2	Scanning a document	18
6.3	Scanning and appending.	18
6.4	Opening a file	18
6.5	Saving a file.....	18
6.6	Closing a text window.....	18
6.7	Closing the application	19
6.8	Starting reading	19
6.9	Pausing reading	19
6.10	Stopping reading	19
6.11	Reading highlighted.....	19
6.12	Spelling highlighted.....	19
6.13	Preceding word	19
6.14	Next word	19
6.15	Starting reading at marker.....	19
6.16	Changing text size.....	20
6.17	Inverting text.....	20
6.18	Word scrolling	20
6.19	Multiple document layout.....	20

6.20 Changing menu size.....	20
6.21 Common MS Windows commands.....	21
7. SETTINGS	23
7.1 Scanner Settings	23
7.2 OCR settings	23
7.3 Speech Settings	24
7.4 Application Settings.....	25
7.5 Keyboard Shortcuts.....	26
8. KEYBOARD SHORTCUTS.....	27

LVI Low Vision International AB was founded in 1978 and is one of the world's leading manufacturers of magnifying TV and computer systems for visually impaired people. New products are developed in close cooperation with users and vision rehabilitation professionals. LVI is certified to Quality System ISO 9001 and Environmental System ISO 14001.

The information in this manual has been carefully checked and is believed to be accurate. However, LVI assumes no responsibility for any inaccuracies that may be contained in it. In no event will LVI be liable for direct, indirect, special, exemplary, incidental, or consequential damages resulting from any defect or omission in this manual. In the interests of continued product development, LVI reserves the right to make improvements in this manual and the products it describes at any time, without notice or obligation.

1. Introduction

MagniLink Read (MLREAD) is a program used in conjunction with a flatbed scanner or a camera Scanner to scan an image and convert it into speech using an OCR unit. The text is then converted to speech using a text-to-speech engine.

1.1 System Requirements*

PC:

- Microsoft® Windows® XP with Service Pack 2 (SP2):
1, 6 GHz processor, 512 MB RAM.
- Microsoft® Windows® Vista or Microsoft® Windows® 7:
2, 0 GHz processor, 1024 MB RAM.

Scanner:

- Twain or Wia-compatible scanner.

* LVI has tested MagniLink Read on several computers and with several scanners without problems. However, this is no guarantee that every combination of computers and scanners will work properly, even those meeting the requirements. LVI recommends testing the intended scanner and computer to ensure compatibility.

2. Installation

2.1 Software download

If no DVD is available, MagniLink Read can be downloaded from the Internet: <http://www.lvi.se>

Select language and press GO.

Go to "Support" and "Download area".

If a language other than English was selected above, click the link "Visit our English site to download trial versions" and select MagniLink Read.

Fill in the form with contact information and press "Send". An e-mail will be returned to the e-mail address stated in the contact information form with a link to the download site.

Click the link and "Download" arrow.

Select "Open" to install or "Save" for later installation.

2.2 Installation

N.b.


The installation requires administrator's rights on the computer and the installation must be run in the context of the intended user.

First install the scanner according to the scanner manufacturer's instructions.

Insert the **MagniLink S Software** CD into your CD-ROM drive (or Download from the Internet, see: 2.1). Your installation should start in a few seconds. If the Auto run function does not start, type "**D:\autorun**" under the command "**Run**" in the Start menu, if "**D:**" is the letter for your CD-ROM drive.


When the installation program starts, follow the on-screen instructions to complete the software installation. See below for a more detailed description.

If Windows needs updating before the installation, a window will pop up.

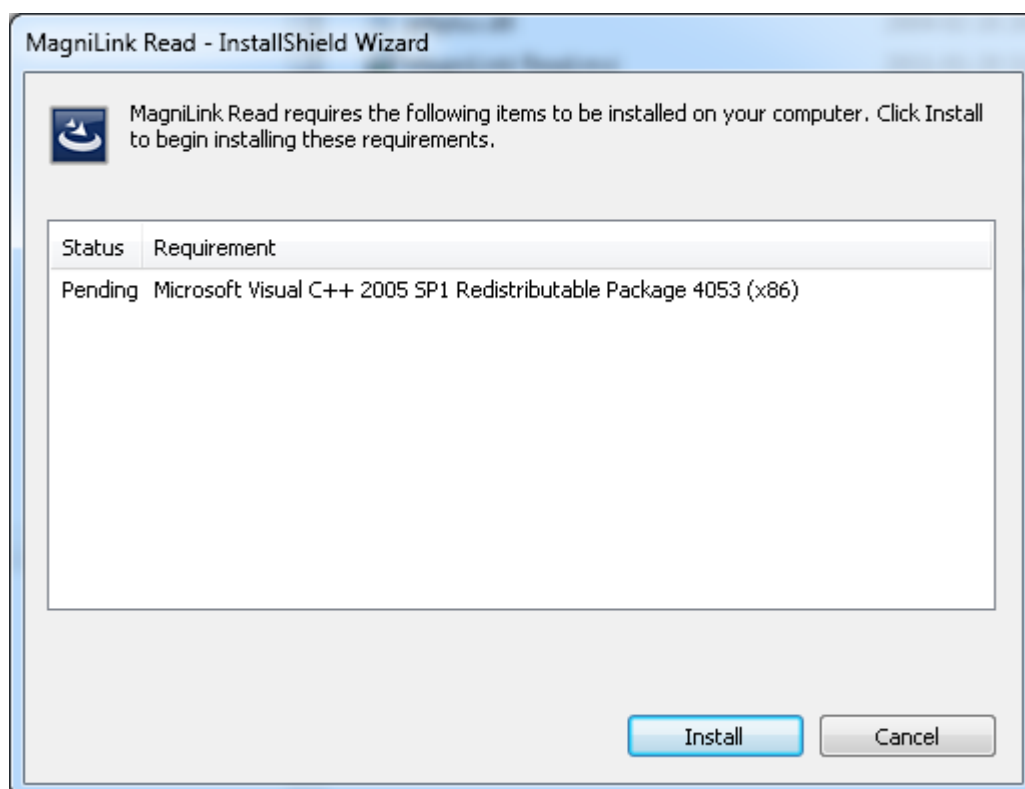


Select **"Install"** and select **"Yes"** in the following window to accept Microsoft's licence terms.


When all necessary Windows components are installed, the installation for MagniLink Read will start and a Welcome window will open.

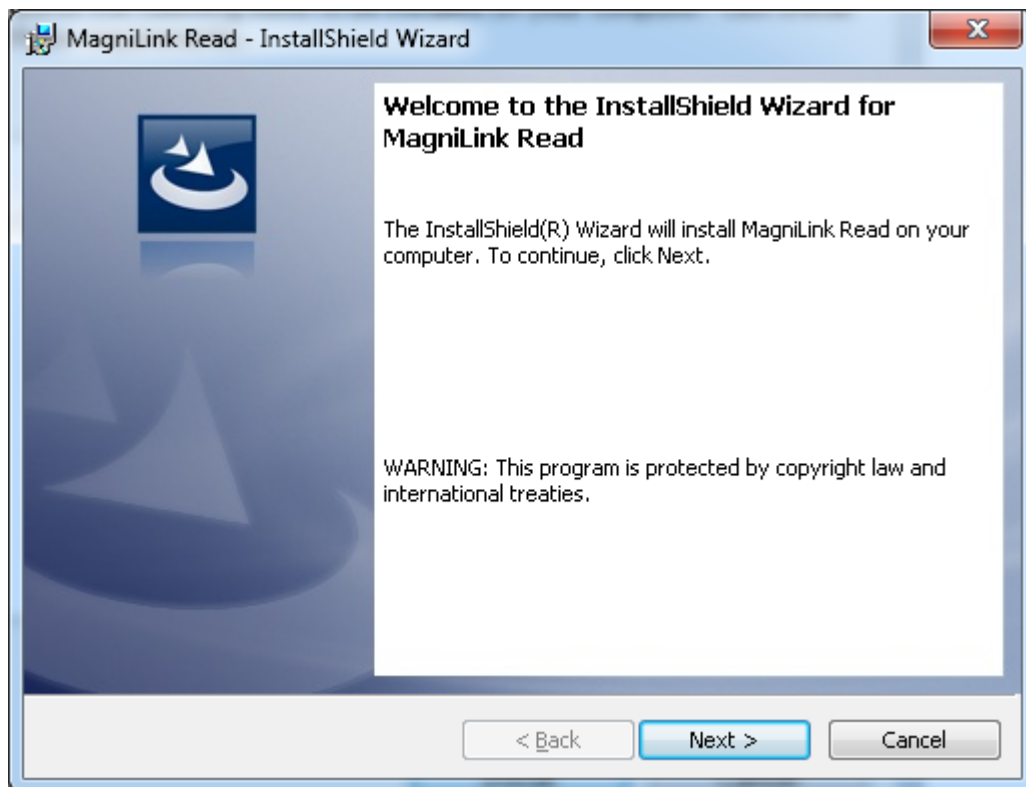


In the dialogue box **"Welcome to the InstallShield Wizard for MagniLink Read"**, click **"Next"**.

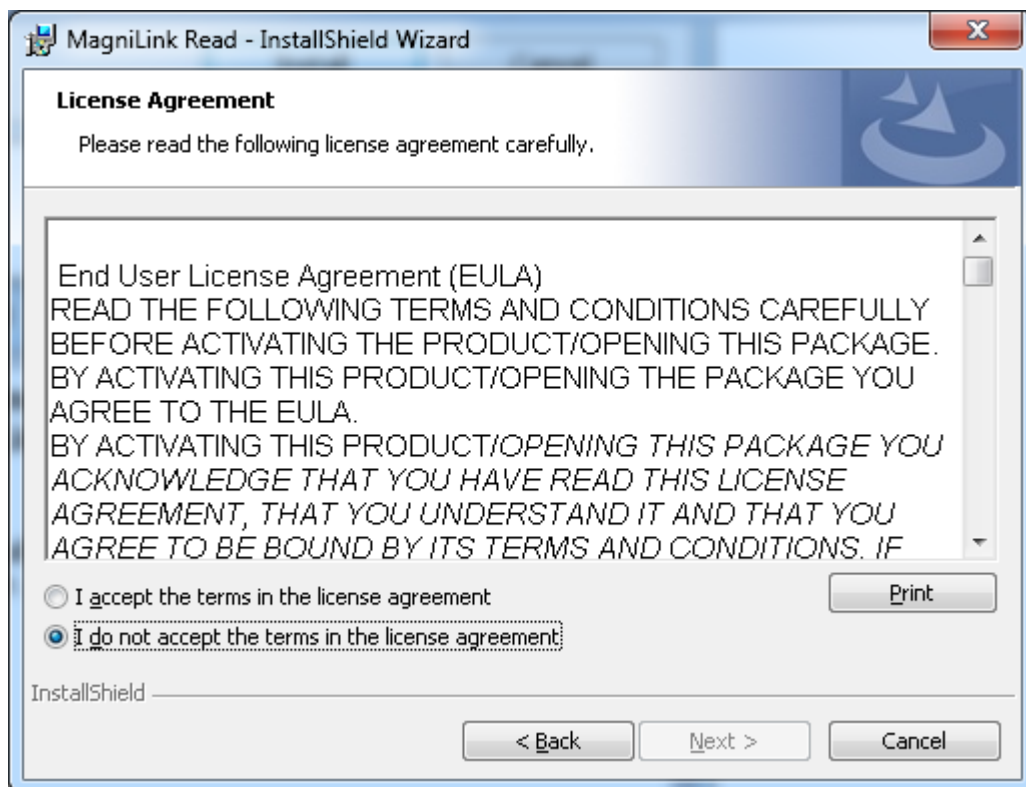

In the dialogue box "**Licence Agreement**", read the text and agree by clicking the appropriate radio button and clicking "**Next**".

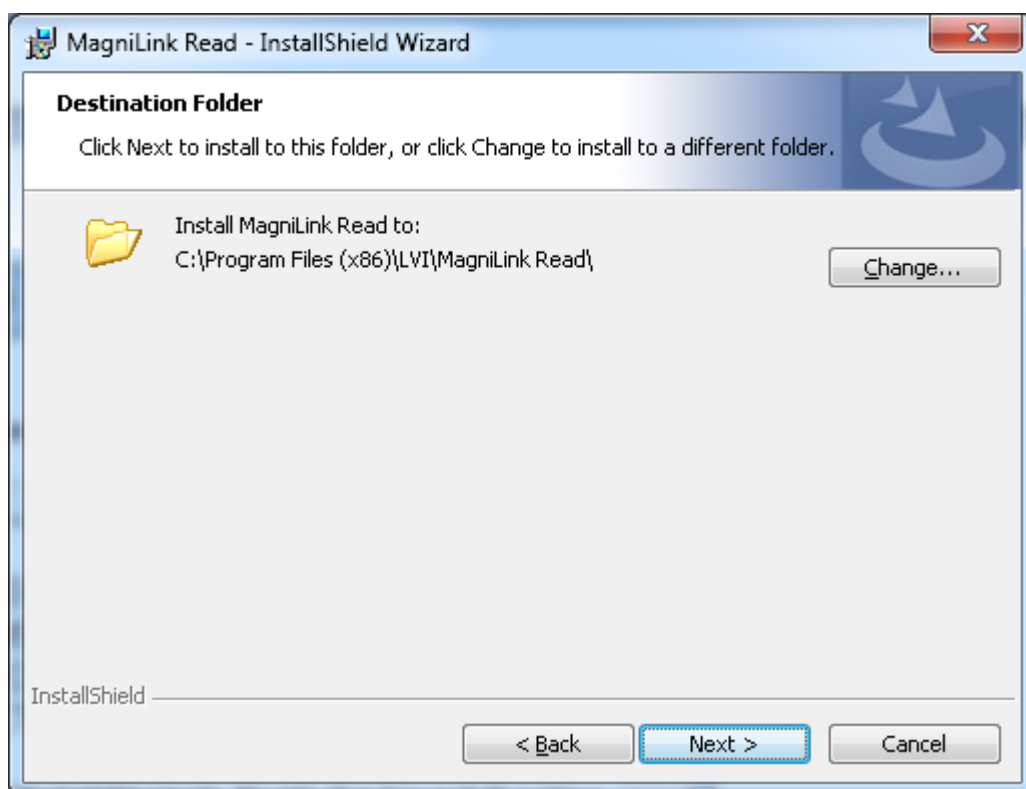


In the dialogue box **"Choose Destination Location"**, click **"Next"** to install the program in the default directory. It is also possible to choose a new installation directory for the program. If the default destination is selected, the program will be installed in an LVI directory with other LVI programs on the computer.

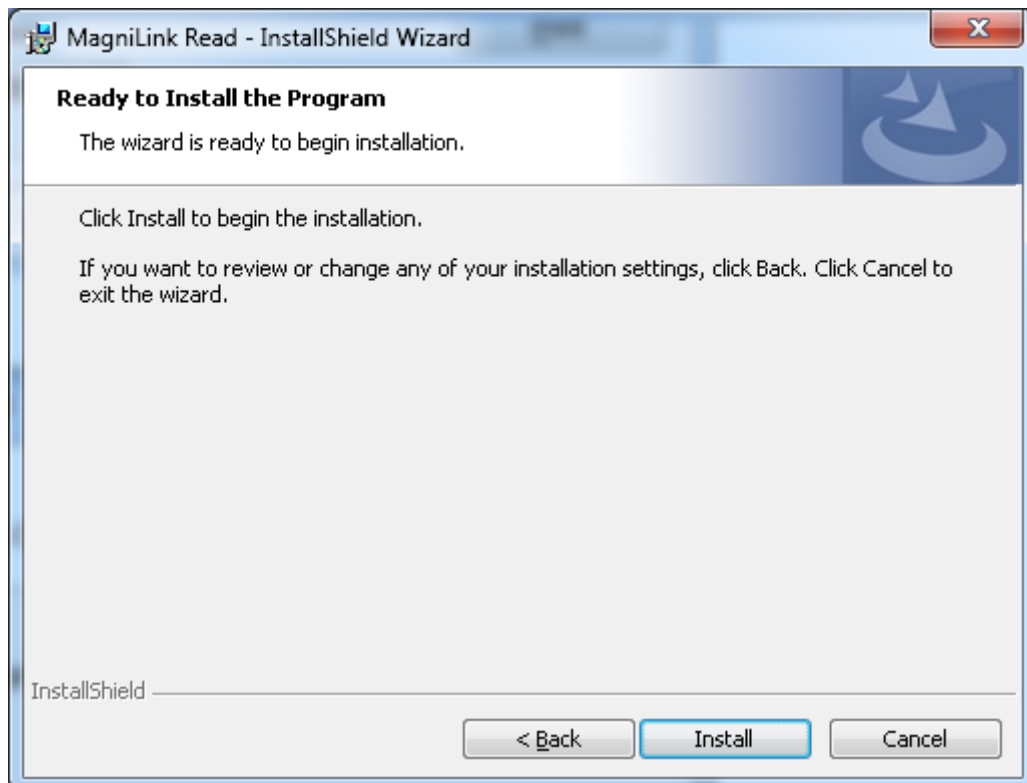

In the dialogue box **"Ready to Install the Program"**, click **"Install"**.

The installation starts and may take a minute or two to complete.

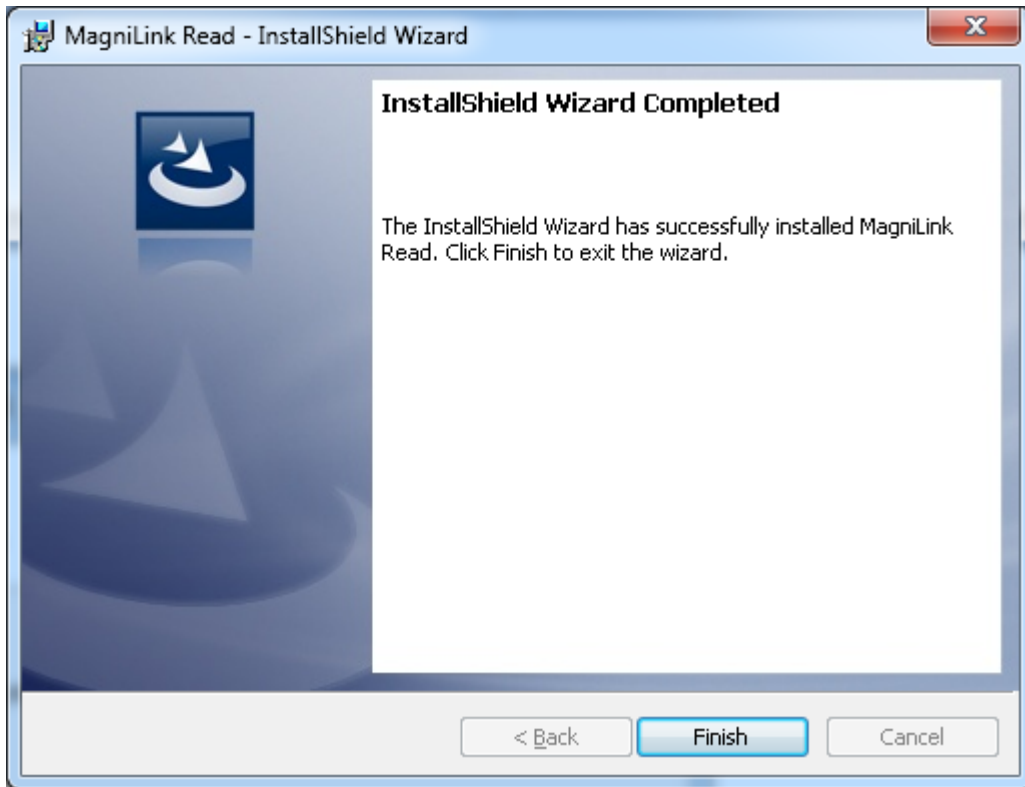


In the dialogue box **"InstallShield Wizard Complete"**, click **"Finish"**.

Start the software from the **"MagniLink Read"** icon at the computer desktop or from the Windows Start menu.

3. Licensing

MagniLink Read needs to be activated before it can be used with its full functionality. The evaluation version of MagniLink Read is limited to 30 days.

When you start a non-activated version of MagniLink Read, the following dialogue will appear. You can choose if you want to activate the software or continue using it for the time-limited period.

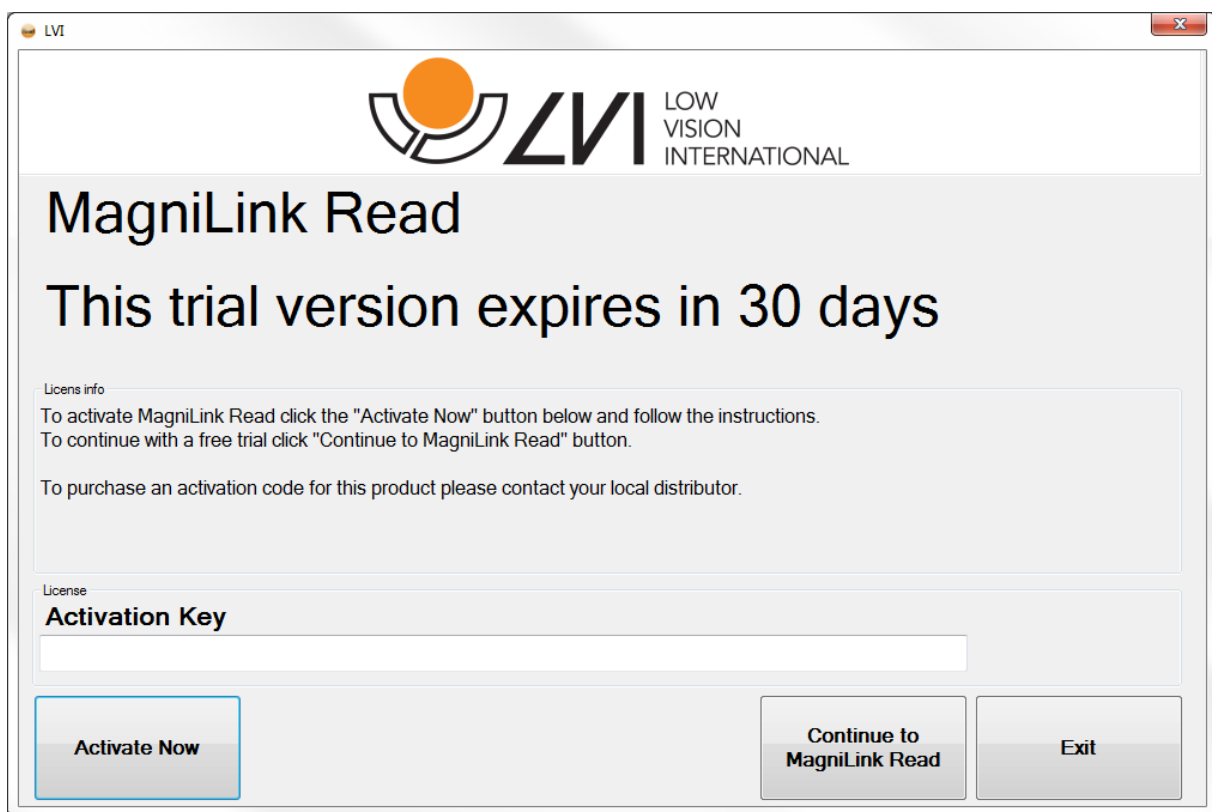


To activate MagniLink Read software, use the following procedure:

Write the activation key in the activation key field and click the button "Activate Now". See above.

If no activation key is available, contact your local LVI dealer to buy a licence and activation key.

3.1 Connected to the Internet

If the computer is connected to the Internet and if activation key has been correctly entered, a new activation dialogue will appear in which we ask for some contact information. Name and e-mail address are mandatory.

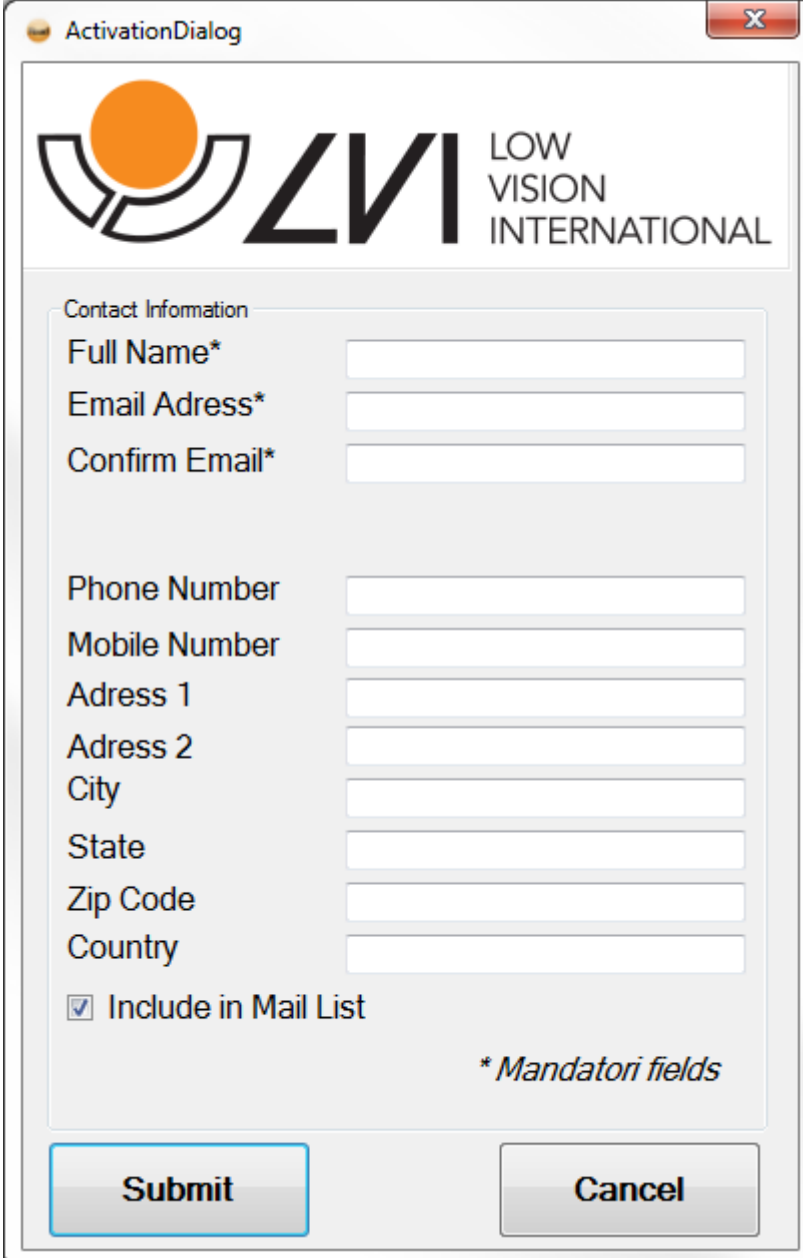

The image shows a software window titled "ActivationDialog" with a close button (X) in the top right corner. The window contains the LVI logo and the text "LOW VISION INTERNATIONAL". Below this, there is a section titled "Contact Information" containing several input fields and a checkbox. The fields are labeled: "Full Name*", "Email Address*", "Confirm Email*", "Phone Number", "Mobile Number", "Adress 1", "Adress 2", "City", "State", "Zip Code", and "Country". The checkbox is labeled "Include in Mail List" and is checked. At the bottom right of the form area, there is a note: "* Mandatori fields". At the bottom of the window, there are two buttons: "Submit" and "Cancel".

Contact Information	
Full Name*	<input type="text"/>
Email Address*	<input type="text"/>
Confirm Email*	<input type="text"/>
Phone Number	<input type="text"/>
Mobile Number	<input type="text"/>
Adress 1	<input type="text"/>
Adress 2	<input type="text"/>
City	<input type="text"/>
State	<input type="text"/>
Zip Code	<input type="text"/>
Country	<input type="text"/>
<input checked="" type="checkbox"/> Include in Mail List	

* Mandatori fields

Press "Submit" to finish the activation.

3.2 Not connected to the Internet

If the computer has no access or not is connected to the Internet when activating, another dialogue will appear showing a computer-generated ID.

Contact your local LVI dealer and present the activation key you received when purchasing the licence, your personal contact information and the computer-generated ID.

You will then be given a new activation code from which the licence can be activated without Internet access.



4. Auto update

MagniLink Read is prepared for automatic update via the Internet, enabling you to obtain the latest updates and always have current software.

Under the "Settings" tab "Application Settings" (see 7.4), you can decide whether or not the application should search for updates every time it starts.

Under the "Settings", tab "Application Settings" (see 7.4), there is also a button allowing you to search for new updates at any time during program execution.

N.b.

During updates, some anti-virus programs may show a warning indicating that the update client is trying to connect to the Internet. Chose "Allow" in order for the update to complete.

It is often possible to disable further warnings for MagniLink READ Auto update in the warning dialogue.


5. User Interface


The User Interface is of the "Multiple Document Interface" type, which means several text windows can be opened at once.

Before text has been scanned or a text window opened with a text file, most of the user interface commands are hidden.

Start by scanning a document (see: 6.2) or open a text file.

When scanning, the process dialogue gives information about the scanning and the OCR conversion progress.

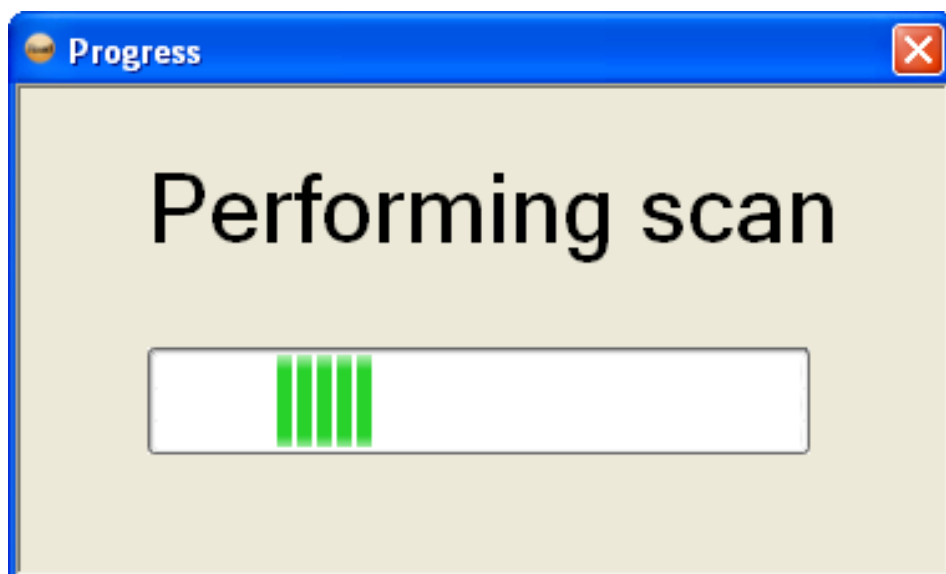

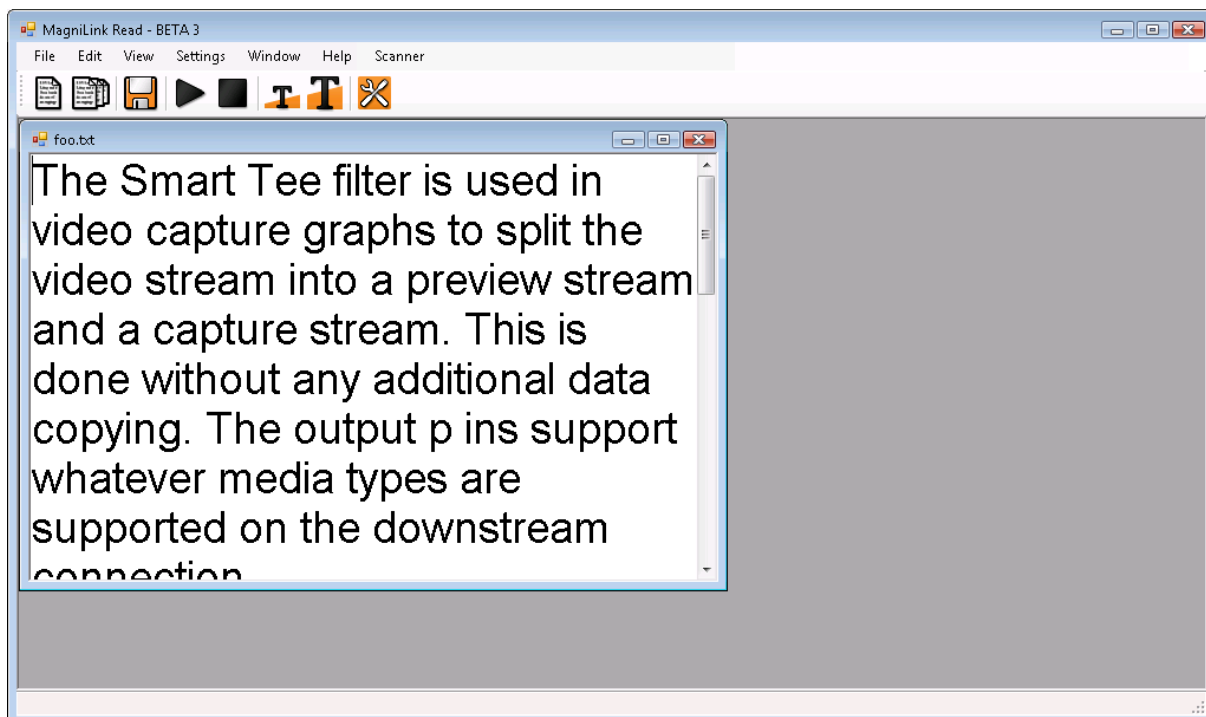

When a text window is open, further commands and menus are accessible.

After the first scan, a new text window is opened for every new document scanned (see 6.2), unless "Scan and Append" (see 6.3) is used.

In this case, the new text will be appended to the actual text in the most recently used text window.

5.1 Command icons



Scan to a new text window
See. 6.2



Scan and append
See. 6.3



Save a file
See. 6.5



Start reading
See. 6.8



Pause reading
See. 6.9

The pause icon replaces the start icon during reading.



Stop reading
See. 6.10



Reduce text size
See. 6.16



Enlarge text size
See. 6.16



Settings
See 7.

6. Working with MagniLink Read

6.1 Starting the program

The program can be started by clicking the desktop icon or through the Windows Start menu. The menus can be accessed by pressing the **Alt** key. The menu items are then showed with their respective shortcuts underlined.

6.2 Scanning a document

To scan a document into a new text window, click the "**Scan**" icon or by selecting "**File -> Scan**". Default keyboard shortcut: **F2**. Depending on the "Application Settings", the speech will start automatically when scanning and OCR conversion have finished.

6.3 Scanning and appending.

To append the text into an open text window, click icon "**Scan and Append**" or by selecting "**File -> Scan and Append**". Default keyboard shortcut: **F3**.

6.4 Opening a file

Files can be opened by selecting "**File -> Open**" or by the mouse dragging the file and dropping it in the applications main window. The files can be either text files (.txt), Word files or PDF files up to version 1.4 and also image files. To open a Word file a Microsoft Word installation is required on the computer. If the user opens an image file, it will immediately be processed by the OCR engine and presented as text. Default keyboard shortcut: **Ctrl + O**.

6.5 Saving a file

An open text window can be named and saved as a file with the icon "**Save As**" or by selecting "**File -> Save As**". The program can only save text files (*.txt). Default keyboard shortcut: **Ctrl + S**.

6.6 Closing a text window

An open text window can be closed by using default the keyboard shortcut: **Ctrl + F4**.

6.7 Closing the application

The application can be shut down by selecting **"File -> Exit"**. Default keyboard shortcut: **Ctrl + Q**.

6.8 Starting reading

When the application has a text document opened, speech can be started by clicking the **"Start Reading"**, by selecting **"Edit -> Start Reading"** or by using the default keyboard shortcut, **Ctrl + 1**. If reading is in "Pause mode", it will resume from the position where it was paused. If reading has been stopped, it will start again from the beginning of the text.

6.9 Pausing reading

To pause reading, click the **"Pause Reading"** button or select **"Edit -> Pause Reading"**. Default keyboard shortcut: **Ctrl + 2**.

6.10 Stopping reading

Speech can be stopped by clicking icon **"Stop Reading"**, or select **"Edit -> Stop Reading"**. Default keyboard shortcut: **Ctrl + 3**.

6.11 Reading highlighted

Read highlighted words by selecting **"Edit -> Read Highlighted"**. Default keyboard shortcut: **Ctrl + 4**.

6.12 Spelling highlighted

Spell a highlighted word by selecting **"Edit -> Spell Highlighted"**. Default keyboard shortcut: **Ctrl + 6**.

6.13 Preceding word

Step the highlight marker one word backward and read that word by selecting **"Edit -> Preceding Word"**. Default keyboard shortcut: **Ctrl + 5**.

6.14 Next word

Step the highlight marker one word forward and read that word by selecting **"Edit -> Next Word"**. Default keyboard shortcut: **Ctrl + 7**.

6.15 Starting reading at marker

Highlight a word and start reading from this word by selecting **"Edit -> Start Reading at Marker"**. Default keyboard shortcut **Ctrl + 8**.

6.16 Changing text size

The text size can be enlarged by clicking the icon **"Enlarge Text Size"** and reduced by clicking the icon **"Reduce Text Size"**. Default keyboard shortcuts are **Ctrl + (+)** to enlarge text size and **Ctrl + (-)** to reduce it.

6.17 Inverting text

To exchange the text and background colours, select **"View -> Invert Text"**. Default keyboard shortcut **Ctrl + I**.

6.18 Word scrolling

Switch to word scroll mode by selecting **"View -> Word Scroll"**. One word at a time will then be displayed at the centre of the screen and the word will be read if speech is on. Resume standard mode by selecting **"View -> Word Scroll"** again. Default keyboard shortcut: **F4**.

6.19 Multiple document layout

The user can change the layout of the documents.

- To tile the documents horizontally, select **"Windows -> Tile Horizontal"**. Default keyboard shortcut **Ctrl + F5**.
- To tile the documents vertically, select **"Windows -> Tile Vertical"**. Default keyboard shortcut **Ctrl + F6**.
- To arrange the documents in a cascade, select **"Windows -> Cascade"**. Default keyboard shortcut **Ctrl + F7**.
- To minimise all documents, select **"Windows -> Minimise All"**. Default keyboard shortcut **Ctrl + F8**.
- To maximise the active document, select **"Windows -> Maximise Active"**. Def. keyboard shortcut **Ctrl + F9**.

6.20 Changing menu size

Select small or large icons and menu text via **"View -> Menu Size"**. Default size is large.

6.21 Common MS Windows commands

The window can be controlled with the mouse or by standard MS Windows commands. The most common Windows commands in connection with ML Read are presented below. For more information about MS Windows, please consult your Windows user manual.

Copy text

When a text window is open, some or all of the (highlighted) text can be copied to, say, a word processor by selecting **Edit -> Copy**. Default keyboard shortcut **Ctrl + C**.

Pasting text

When a text window is open, text copied or cut from other programs can be pasted for reading by selecting **"Edit -> Paste"**. Default keyboard shortcut **Ctrl + V**. It is also possible to mark text in another program and by the mouse drag and drop the text in the applications main window.

Switching between open applications:

Press **"Alt" + "Tab"**

Switch between full screen (maximised window) and normal window.

With the mouse:

Click on the **"Restore"** icon in the upper right corner of the window (between maximise and minimise).

With the Keyboard:

Press **"Alt" + "Spacebar"** and the application control menu opens. Choose **"Restore"** in the menu to go to normal window. Choose **"Maximise"** to go to full screen.

Change the size / position of the window.

Make sure you are in the normal window mode (see above).

With the mouse:

Use the mouse to drag each side of the window to the preferred position.

With the Keyboard:

Press **"Alt + Space"** and choose **"Change size / Move"** in the application control menu. Set the preferred size or position of the window using the arrow keys.

Example:

To change the height of the window: Start by pressing the "down arrow" key in order to move the bottom of the window and set the preferred position with the "up arrow" key. Press "Enter" to finish.

To move the window horizontally: Start by pressing the "left arrow" key or "right arrow" key to move the window left or right. Press "Enter" to finalise settings.

7.Settings

The settings dialogue can be accessed from the menu. Select **Settings** and some of the **Scanner, OCR, Speech, Application** or **Keyboard Shortcuts**. The settings dialogue can also be accessed via the "**Settings**" icon. Default Keyboard shortcut **F9**.

7.1 Scanner Settings

Scanner:

Selects one particular scanner when more than one is connected to the computer.

Resolution:

Enables a change of resolution of the scanned document to "Standard" or "High". "Standard" resolution is a compromise between OCR demands, speed and file size and will work in most cases. If very small text is to be scanned, "High" resolution may give better results but at the expense of slower scanning and use of more memory.

Default:

This will return the scanner settings to their factory defaults.

Note:

With "**Default All**" you can restore all settings to factory defaults.

7.2 OCR settings

OCR stands for "Optical character recognition" and is a process by which an image of text is translated into editable text.

Highlight Errors:

When this checkbox is selected, the OCR engine will highlight uncertain words in a different colour. See below.

Error Highlight Colour:

Colour selection for words that the OCR engine marks as uncertain.

OCR- Language:

This combo box allows you to change the language of the OCR engine. For best results, it is important that the OCR engine language is the same as that of the document being processed.

Fast Mode:

When this checkbox is selected, the OCR Engine will work faster but less accurately.

Default:

This will return the OCR settings to factory defaults.

7.3 Speech Settings

This tab allows you to change voices and voice parameters for reading text and commands.

Speed:

This parameter sets the speed of the voice.

Volume:

This slider sets the sound level of the voice.

Voice:

If more than one voice is installed, the user will be able to choose from them.

Default:

This will return the speech settings to factory defaults.

7.4 Application Settings

This tab allows the user to change some behaviours of the application.

Start Speech after OCR:

This checkbox allows the user to set whether speech should start automatically immediately after the OCR process.

Read Commands:

This checkbox allows the user to set whether menu commands and other commands should be spoken by the application. It is a good idea to disable this setting if the user already has a screen reader.

Look for updates at startup:

When this checkbox is checked the update manager will automatically search for new updates at every startup. See also "Auto Update"

Look for updates:

Pressing this button will start an immediate search for new updates.

Select Font:

This button opens a dialogue used to set the font for the document text and word scroll.

Set Text Colour:

This button opens a colour dialogue allowing the user to set the colour of the text.

Set Background Colour:

This button opens a colour dialogue that allows the user to set the background colour.

Default:

This will return the application settings to factory defaults.

7.5 Keyboard Shortcuts

This dialogue allows you to change the keyboard shortcuts. To change a shortcut, use the following procedure.

1. Browse to the command where you want to change the shortcut and press the **Enter** key. This will open up a new dialogue containing three checkboxes, a combobox, an **OK** button and a **Cancel** button.
2. Use the **Tab** key to toggle between the controls. When one of the modifier checkboxes is active, press the **Space** key to set or unset that modifier.
3. Use the **Tab** key to switch to the combobox. Select the new shortcut key with the arrow keys.
4. Tab to the **OK** button and press **Enter**. If the selected key combination is invalid, a dialogue box with an error message will appear and the procedure must be repeated.

Default:

This will set the **Keyboard Shortcuts** back to factory default.

8.Keyboard Shortcuts

Scan a Document	F2
Scan and Append	F3
Open	Ctrl + O
Save As	Ctrl + S
Close Window	Ctrl + F4
Exit Application	Ctrl + Q
Start Reading	Ctrl + 1
Pause Reading	Ctrl + 2
Stop Reading	Ctrl + 3
Read Highlighted	Ctrl + 4
Spell Highlighted	Ctrl + 6
Preceding Word	Ctrl + 5
Next Word	Ctrl + 7
Start Reading at Marker	Ctrl + 8
Enlarge Text Size	Ctrl + "+"
Reduce Text Size	Ctrl + "-"
Word Scroll	F4
Invert Text	Ctrl + I
Settings	F9
Tile Horizontal	Ctrl + F5
Tile Vertical	Ctrl + F6
Cascade	Ctrl + F7
Minimise All	Ctrl + F8
Maximise Active	Ctrl + F9