# Rengöringsindikatorer

**Andreas Bengtsson**

**Getinge Sverige AB**

[andreas.bengtsson@getinge.com](mailto:andreas.bengtsson@getinge.com)

**010-335 31 06**

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## Standard

### **ISO15883:**

“Routine test; periodic checking and testing carried out to establish that the operational performance of the washer disinfectors remains within the limits established during validation“

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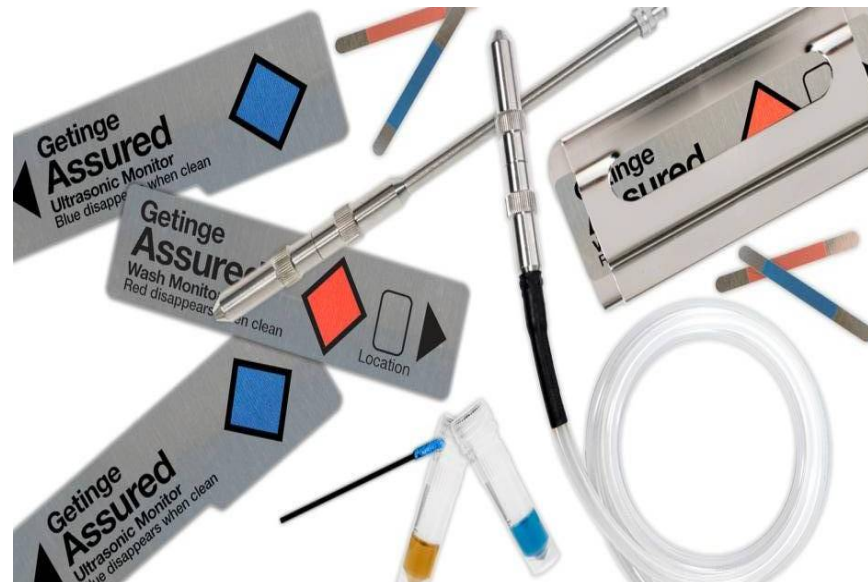
### **AAMI:**

ST79 10.2 – *“To ensure that mechanical cleaning equipment is working properly, and according to the manufacturer’s specifications, health care personnel may perform verification tests as part of the overall quality assurance program. **This verification may include the use of test devices that monitor the functionality** of the cleaning equipment in cleaning surfaces and that ensure adequate fluid flow in equipment that has adaptors for lumened devices.”*

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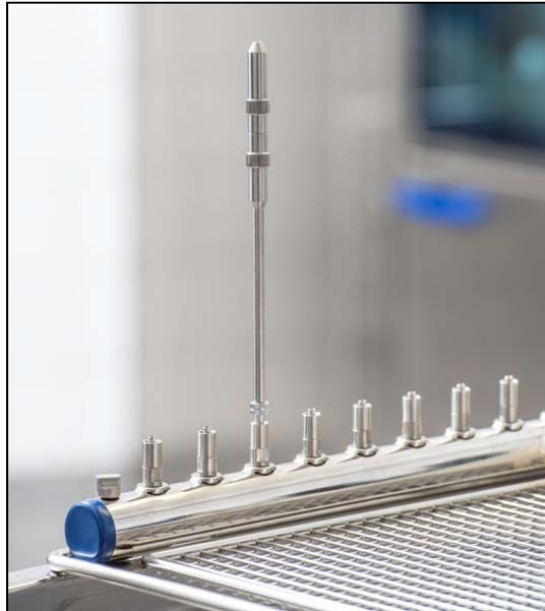
Wash Monitor F/R

## Getinge Assured Ultrasonic Monitors

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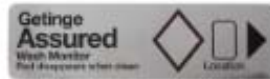
**INTERPRETATION CHART  
GETINGE ASSURED WASH MONITORS**


**WASH MONITOR**


UNEXPOSED

PASS

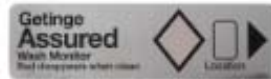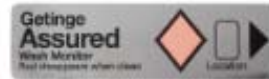**FAIL**  
Detergent: Insufficient  
Impingement: Insufficient

**FAIL**  
Detergent: Insufficient  
Impingement: Insufficient

**FAIL**  
Detergent: Insufficient  
Impingement: Sufficient

**WASH MONITOR F/R • FLEXIBLE & RIGID**

UNEXPOSED

PASS

FAIL

FAIL

**WASH MONITOR U**

UNEXPOSED

**FAIL**  
Check the ultrasonic  
equipment and water and  
detergent used.

PASS

**WASH MONITOR U/L**

UNEXPOSED

PASS

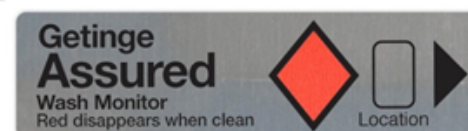
**FAIL**  
Check the ultrasonic equipment and  
water and detergent used.

FAIL

# Protokollet

## GETINGE ASSURED WASH MONITOR DOKUMENTERING

Datum/Sign.	Lastinformation	Resultat	Häfta Wash Monitor Nedan
Datum: __/__/__ Sign: _____	Maskin: _____ Cykel #: _____ Plan #: _____ Lot #: _____	<input type="radio"/> OK <input type="radio"/> Ej OK <input type="radio"/> Marginellt	
Datum: __/__/__ Sign: _____	Maskin: _____ Cykel #: _____ Plan #: _____ Lot #: _____	<input type="radio"/> OK <input type="radio"/> Ej OK <input type="radio"/> Marginellt	
Datum: __/__/__ Sign: _____	Maskin: _____ Cykel #: _____ Plan #: _____ Lot #: _____	<input type="radio"/> OK <input type="radio"/> Ej OK <input type="radio"/> Marginellt	

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